

SaniSorb[®]

Medical Liquid Solidifier

Advantages

- Economical
 - pre-measured, dissolvable pouches
- Easy to use, self-bursting pouch
 - no pouring or tearing needed
- Particulate-free
 - eliminates harmful and annoying dust
- Reduced exposure
 - minimizes risk of cross contamination of infectious waste to medical staff

Applications

- Suction Canisters
- Red Bag Waste
- Kick Buckets
- Medical Containers
- Drainage Systems

SaniSorb Medical Liquid Solidifier

SaniSorb medical liquid solidifier pouches are the first liquid solidifier with self-bursting technology to quickly absorb and immobilize fluid. SaniSorb pouches provide fast and easy infectious waste fluid containment in operating room suction canisters and red bag waste by quickly solidifying biohazardous fluids. SaniSorb pre-measured rupturing pouches eliminate the tearing and spillage associated with conventional products. Self-activating technology reduces particulates and bacterial exposure. Available in four standard sizes, the pouches dissolve within minutes on contact with fluid to quickly absorb and immobilize fluid.

SaniSorb Pouches Absorption Chart Per 15 gram Pouch

Canister Size (cubic centimeters (cc))	No. of Pouches
500	1
1000	2
1500	3
2000	4
2500	5
3000	6

SaniSorb 15 gram Pouch Usage Directions

- Wear gloves and eye protection when handling infectious waste.
- Insert the SaniSorb pouches through the pour spout of the suction canister. See table for applicable number of pouches depending on size of suction canister.
- Recap pour spout and allow for solidification before handling.
- Dispose of treated medical waste in compliance with local, state and federal regulations.

SaniSorb pouches eliminate the need to pour loose powder, thereby reducing dusting and spillage.



All information provided by Multisorb Technologies, Inc. ("Multisorb") as product literature or through technical assistance service (hereinafter "Information") is believed to be accurate but is not guaranteed as such. It remains the responsibility of the customer to confirm all technical aspects of the application of the products and the product literature including but not limited to compliance with all applicable laws, rules and regulations. Multisorb reserves the right to make additions, deletions or modifications to the Information at any time without prior notice. Information related to Multisorb products is for use by technically skilled persons at their own discretion and risk. The Information is not to be deemed a license under any patent or proprietary right. Multisorb shall not be liable for any incidental, special, indirect, direct, special and/or consequential damages involving the use of their products, product literature and or services.

www.multisorb.com



MULTISORB
Technologies